



**O-GENESIS™**  
GRAFT DELIVERY SYSTEM


## Graft Delivery System



# O-GENESIS™ Graft Delivery System

The O-GENESIS™ Graft Delivery System is a complete bone graft delivery system designed to deliver allograft, autograft or synthetic bone graft to all orthopedic surgical sites. The system is provided sterile and is for single-use only.

## Two Versatile Loading Options

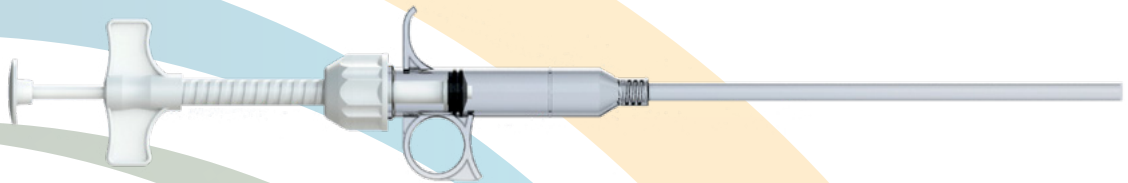

### Syringe and Plunger

The syringe and plunger provide streamlined loading of hydrated and flowable bone graft material into the cannula. This intuitive system is designed to reduce time when preparing bone graft for implantation.



### Funnel

A funnel attachment for the cannula is provided to load granular autograft and allograft, as an alternative to the syringe/plunger.

## Cannula

The transparent cannula allows visibility of graft advancement during loading and delivery. For surgeries requiring more than one cannula, additional cannulas are available.



### Cannula Caps



Cannula caps are provided to maintain hydration of bone graft material in the cannula until ready for use..



## Precise and Controlled Delivery

The ergonomic design of the O-GENESIS Graft Delivery System enables precise and controlled delivery of bone graft material. The ratchet mechanism paired with a curvaceous trigger handle provides seamless graft placement; single handedly delivering a consistent volume.

Graft is visible in the cannula, advancing as the trigger handle is being pulled.

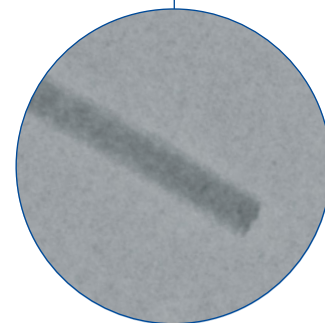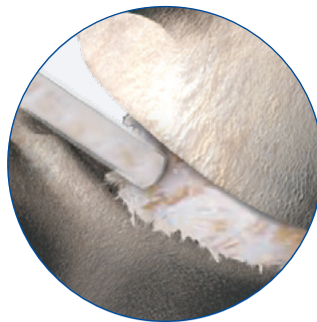


Radiographic image with allograft

## Key Advantages

- One-handed delivery
- Versatile delivery of the bone graft of your choice
- Sleek design to allow greater visibility of the surgical site
- Tangible resistance when the bone void is filled
- Pre-fill or post-pack without repositioning
- Eliminate sterilization cost and logistics

Please visit [Orthofix.com/IFU](http://Orthofix.com/IFU) for full information on indications for use, contraindications, warnings, precautions, adverse reactions and sterilization.

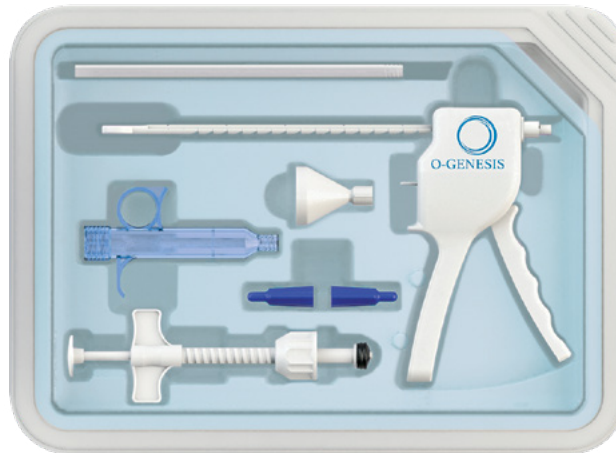

Part Numbers	Description
35-6000SP	O-GENESIS Graft Delivery System
35-6001SP	O-GENESIS Cannula and Caps

Caution: Federal law (USA) restricts this device to sale by or on the order of a physician. Proper surgical procedure is the responsibility of the medical professional. Operative techniques are furnished as an informative guideline. Each surgeon must evaluate the appropriateness of a technique based on his or her personal medical credentials and experience.

 Orthofix  
3451 Plano Parkway  
Lewisville, Texas 75056-9453 USA  
1.214.937.3199  
1.888.298.5700  
[www.orthofix.com](http://www.orthofix.com)

**Rx Only**

Orthofix products or services referenced herein are trademarks or registered trademarks of Orthofix Medical Inc. and its group of companies. Any rights not expressly granted herein are reserved.

1.888.298.5700  
[orthofix.com](http://orthofix.com)

OG-2001 © Orthofix Holdings, Inc. 7/2020

[orthofix.com](http://orthofix.com) |  **ORTHOFIX®**